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The screenshot shows the Agilent eArray website navigation bar with 'Probe' highlighted. Below it, the 'Search Standard Probes' section is active. The search form contains the following fields and options:

- Search Type:** All (dropdown), Exact Search
- Search Term(s):** (text input)
- Species:** H. sapiens (dropdown), * Include subfolders
- Used in Array Designs:** (text input), [Select and Add](#)
- Used in Probe Groups:** (text input), [Select and Add](#)
- miRBase Version:** 18.0 (dropdown), *
- Folder:** AgilentCatalog (dropdown), * Include subfolders
- Include * Sequence:** Include * Sequence

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The screenshot shows the Agilent eArray search interface. The 'Search Standard Probes' section is active. The 'Search Type' dropdown menu is highlighted with a red box, showing the following options: All, ProbeID, miMAT#, microRNA Name, ChromosomalLocation, and ProbeSequence. Other search parameters include 'Exact Search' (unchecked), 'Folder' (AgilentCatalog), 'Include subfolders' (checked), 'Used in Probe Groups' (empty), 'Include * Sequence' (checked), and 'miRBase Version' (empty). There are 'Search' and 'Reset' buttons at the bottom of the search form.

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Search Term(s):

Species [Info](#): H. sapiens *

Used in Array Designs: [Select and Add](#)

miRBase Version: 18.0 *

Folder [Info](#): AgilentCatalog * Include subfolders

Used in Probe Groups [Info](#): [Select and Add](#)

Include * Sequence [Info](#): Include * Sequence

Search Results: 1887 matching results found

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Search Type:	ProbeID	<input checked="" type="checkbox"/> Exact Search
Search Term(s):	A_16_P35089808 A_14_P	Upload

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Search Type:	ProbeID	<input checked="" type="checkbox"/> Exact Search
Search Term(s):		Upload

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https://earray.chem.agilent.com - Ag

Please Enter the Following Details

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Search Type:	ProbeID	<input checked="" type="checkbox"/> Exact Search
Search Term(s):	A_24_P66027IA_32_P771	Upload

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Search

Search Your Probes

Search Type: Exact Search Folder [Info](#) : * Include subfold

Search Term(s): [Upload](#)

Species [Info](#) : * Used in Probe Groups [Info](#) : [Select and Add](#)

Used in Array Designs: [Select and Add](#) Include * Sequence [Info](#) : Include * Sequence

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Search Your Probes

Search Type: Exact Search

Search Term(s): [Upload](#) Folder Info: * Include subfolders

Species Info: * Used in Probe Groups Info: [Select and Add](#)

Used in Array Designs: [Select and Add](#) Include * Sequence Info: Include * Sequence

miRBase Version: *

Search Results: 848 matching results found

[Download](#) [Create Probe Group](#) [Array Calculator](#) Pages [(1)First] 1 2 3 4 5 [Next >>] [Last (17)]

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Home Microarray Probe Group **Probe** My Functions My Account Site Maintenance Data

Search

Search Your Probes

Search Type: All Exact Search Folder [Info](#): Agilent

Search Term(s):

Species [Info](#): H. sapiens * Used in Probe Groups [Info](#):

Used in Array Designs: [Select and Add](#) Include * Sequence [Info](#): Incl

miRBase Version: 14.0 *


Search Results: 887 matching results found

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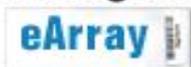Search | [Browse Probe Groups](#)

Create **Probe Group**

Probe Group Name	<input type="text"/>	Created Date	04/14/2010
Status Info	<input checked="" type="radio"/> Incomplete <input type="radio"/> Locked	Description Info	<input type="text"/>
Created by	YAYOI FUKUOKA	Keyword Info	<input type="text"/>
Folder Info	LSCA_JapanSupport	No. of Probes	2446
		Search Criteria Info	Click Here

Search Results: 887 matching results found

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Home | Microarray | **Probe Group** | Probes | My Account | Site Maintenance

Application Type: CGH [Switch Application Type](#)

Search | Browse ProbeGroup

Browse WorkGroup By Category

– Category

Browse Catalog By Category

– Category

- Applications
- Species

Browse By Folder

- ROOT
 - Workspace名
 - AgilentCatalog

View Probe Group

Status:

Search Results: 2 matching results found

<input type="checkbox"/>	Probe Group Name	No. of Probes	High Density	Folder	Status	Created Date	Actions
<input type="checkbox"/>	AllDemo_Rat	99328	true	LSCA_JapanSupportSpace	Incomplete	12-Feb-2008	Copy Edit View Delete Download
<input type="checkbox"/>	test	14680	true	LSCA_JapanSupportSpace	Incomplete	12-Feb-2008	Copy Edit View Delete Download

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
The screenshot shows the Agilent eArray web interface. At the top, there is a navigation bar with tabs: Home, Microarray, Probe Group, Probe, My Functions, My Account, Data, and Primers. On the right, it says "microRNA Switch Application Type". Below the navigation bar is a "Releases & Announcements" section. The main content area is divided into two panels. The left panel is titled "Search" and has radio buttons for "Microarray", "Probe Group", "Probe", and "Primers". It contains input fields for "Microarray Name:", "Species Info:" (with a "Select and Add" link), and "Design ID:" (with an "Upload" link). There are "Search" and "Reset" buttons at the bottom. The right panel is titled "Design Wizards" and has a "View All" link. It shows a radio button selected for "Create a Microarray Design from Existing Probe Group(s)" with an "Info" link. Below this, the "Custom Design Guidance" button is highlighted with a red rectangular box. To the right of this panel, the "Next >>" button is circled in red.

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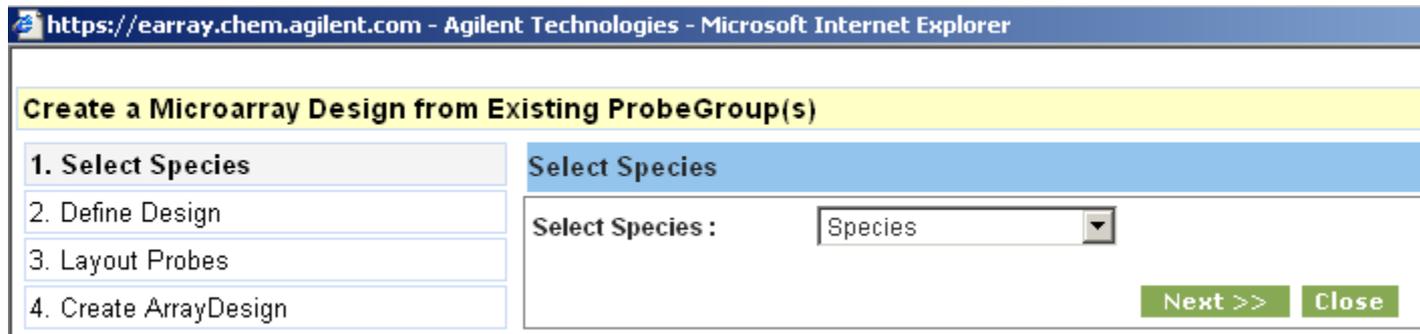Create a Microarray Design from Existing Probe Group(s)

1. Select Species Info	Design Details		
2. Define Design	Microarray Name: <input type="text" value="Test"/>		
3. Layout Probes	Folder: <input type="text" value="Agilent"/>	Species Info : <input type="text" value="H. sapiens"/>	
4. Create Microarray Design	Design Format: <input type="text" value="8 x 60 K"/>	Show Details	Control Grid: IS-62976-8-V2_60Kx8_miR_CPD_20101013_10
	<p>When using 8x60K, it is recommended that you choose 40 features per microRNA for maximum sensitivity. Choosing fewer features may compromise sensitivity.</p>		
	Show Other Design Details: <input type="checkbox"/>		

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① Search

②

Probe Group Name	Folder	Status
Demo	Yayoi	Incomplete
Demonstration	Agilent_Field	Incomplete

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Features per microRNA [Info](#) : 16 **Agilent recommends that you choose 40 features per microRNA for maximum sensitivity. Choosing fewer features may compromise sensitivity.**

Fill Microarrays [Info](#) :

Enable Microarray Set [Info](#)

Probe Group to Fill Microarray : [Structural_FillerProbeGroup](#)

Feature Layout [Info](#) : Randomized

Microarray Statistics:			
Number of Microarrays per Slide:	8	Percentage Filled by Selected Probe Groups:	51.3783%
Number of Slides:	1	<u>Number of Available Features:</u>	<u>30620</u>
Total Number of Features:	62976	Number of User Controls:	0
Number of Agilent Controls:	2164	Number of microRNA Targets Covered:	1887
Total Percentage Filled:	51.3783 %		
Number of Distinct microRNA Targets Covered:	1887		
Number of Distinct microRNA Targets that can be covered by the remaining space:			1913

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
Microarray Statistics:			
Number of Microarrays per Slide:	8	Percentage Filled by Selected Probe Groups:	123.29141 %
Number of Slides:	1	Number of Available Features:	0
Total Number of Features:	62976	Number of User Controls:	0
Number of Agilent Controls:	2164	Number of microRNA Targets Covered:	1887
Total Percentage Filled:	123.29141 %		
Number of Distinct microRNA Targets Covered:	1887		
Number of Distinct microRNA Targets that can be covered by the remaining space:			0

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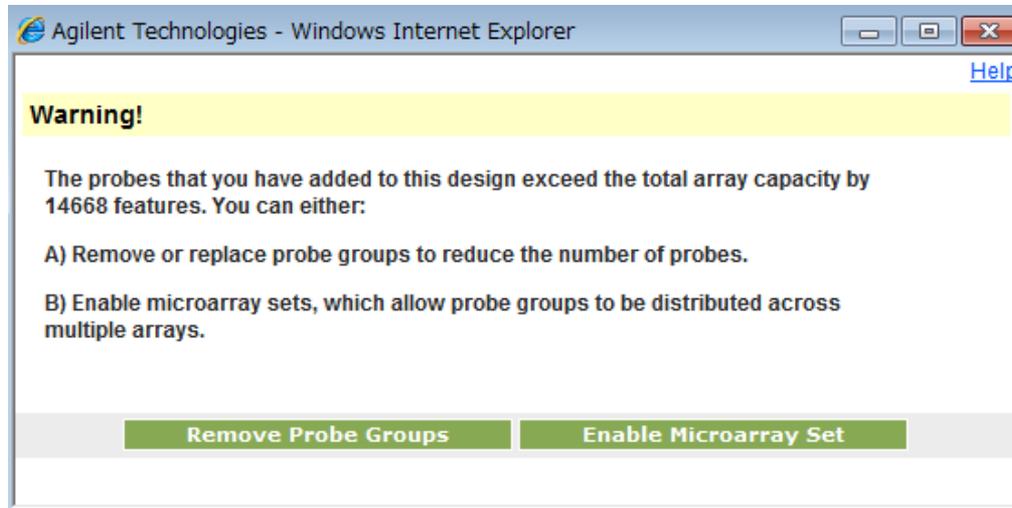Create a Microarray Design from Existing ProbeGroup(s)

1. Select Species
2. Define Design
3. Layout Probes
4. Create ArrayDesign

How do you want to save and create your Design?

<input checked="" type="radio"/>	Draft	Saves the design, allowing only you to make changes to it before completing.	More
<input type="radio"/>	Review	Saves the design, enabling users in your Workgroup to make versions of the design before completing.	More
<input type="radio"/>	Complete	Saves the design, this design cannot be edited, nor can it be ordered.	More
<input type="radio"/>	Submit	Saves the design, this design cannot be edited, and is available for ordering <input type="checkbox"/> Design check list	More

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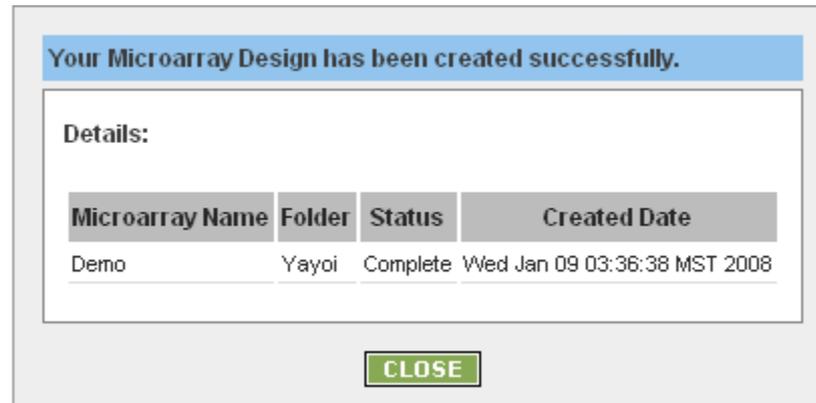
Design Format: Status:

Search Results: 1 matching results found

<input type="checkbox"/>	Microarray Name	Microarrayset name	Folder Name	Status	Design Number	Created Date	Actions
<input type="checkbox"/>	Demo		Yayoi	Complete	0XXXXX	09-Jan-2008	Copy View Delete Submit Download Change Control Type

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Microarray Name: ABC

Status: **Draft** (Draft, Review, Complete)

Folder: [Dropdown]

Species: [Dropdown]

Created By: Yajoi, Erikaoka

Design Format: 4 X 44K [Show Details]

Control Grid: IS-45220-4-V1_4x44K_GX_EQC_V20060608

Description: [Text Area]

Keywords: [Text Area]

Attachment: +/-

Comments: [Text Area]

Feature Layout Info: Randomized Ordered for Agilent Scanner

Select	Probe Group name	Control Type	Replicate	Evenly Distribute
<input type="checkbox"/>	013791_M_mulatta (Rhesus) Oligo Microarray_5	[Dropdown]	5	<input type="checkbox"/>
<input type="checkbox"/>	Human 1A Microarray (V2)	[Dropdown]	2	<input type="checkbox"/>

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Download If you have difficulty downloading the desired file, hold down the <Ctrl> key until a File Download dialog box appears. This bypasses pop-up blocking software.

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Design Format: Status:

Search Results: 1 matching results found

<input type="checkbox"/>	Microarray Name	Microarrayset name	Folder Name	Status	Design Number	Created Date	Actions
<input type="checkbox"/>	Demo		Yayoi	Complete	OXXXXX	09-Jan-2008	Copy View Delete Submit Download Change Control Type

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Warning : You are about to submit this Microarray Design. Once saved no changes can be made


Please enter comments in the textarea:

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The screenshot shows a web interface for submitting a microarray design. On the left is a checklist titled "Did you:" with ten items, each preceded by an unchecked checkbox. A "Help" link is visible in the top right. On the right, a warning dialog box is open, titled "Warning : You are about to submit this Microarray Design. Once saved no changes can be made". It contains a text area with "ANY TEXT" and a "Design check list" checkbox which is checked. At the bottom of the dialog are "Yes" and "No" buttons. Below the checklist, there are several paragraphs of text explaining the submission process and the user's responsibility. At the bottom of the main form area are "Done" and "Cancel" buttons.

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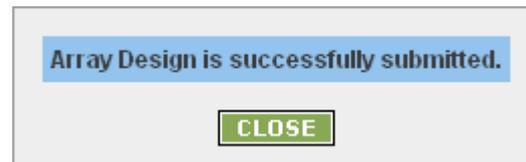 **Agilent Technologies**

You have submitted a design to manufacturing. However, an order needs to be placed for the synthesis to start. To place an order please use the request quote function on the eArray website. After doing so, you will receive a formal quote with further instructions to call in your PO number.

The details of the design are listed below:
 Array Set Name = Demo
 Design Number = 018940
 Slide Format = 8 X 15K
 Application = CGH
 Customer Design ID = NeAT_design
 Slide Layout = IS-15744-8-V1
 Submission Date = 09-Jan-2008 04:06

The details of the Customer are listed below.

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Copyright Agilent Technologies, Inc. 2002-2010