# Design ID/ELIDの確認法

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The screenshot shows the eArray web interface. At the top right, the 'Application Type' is set to 'Expression', with a 'Switch Application Type' link next to it. The navigation menu includes 'Home', 'Microarray', 'Probe Group', 'Probes', and 'My Account'. The 'Microarray' tab is active, and the 'Search' button is highlighted with a red box and circled number 3. Below the navigation menu, there are links for 'Create Microarray', 'Browse ArrayDesign', 'Browse ArraySet', 'Express Array', 'Notification', 'Array Calculator', and 'Print Array'. The main content area is titled 'Microarray Design' and 'Microarray Set'. The search form contains the following fields: 'Microarray Name' (Whole Human Genome), 'Species' (with a 'Select and Add' link), 'Design Status' (radio buttons for All, Active, Obsolete), 'Created Date' (From and To date pickers), 'Keywords' (text input), 'Folder Name' (dropdown menu with 'All' selected and 'Include subfolders' checkbox), 'Design Number' (text input with 'Upload' link), 'Created By' (text input), and 'Containing probe group' (text input with 'Select and Add' link). At the bottom of the form, there are 'Search' and 'Reset' buttons. The 'Search' button is highlighted with a red box and circled number 5. A circled number 4 is placed next to the search form fields.

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Home **Microarray** Probe Group Probes My Account Application Type: **Expression** [Switch Application Type](#)

[Search](#) [Create Microarray](#) [Browse ArrayDesign](#) [Browse ArraySet](#) [Express Array](#) [Notification](#) [Array Calculator](#) [Print Array](#)

**Microarray Design**  **Microarray Set**

Microarray Name: <input type="text" value="Whole Human Genome"/>	Folder Name: <input type="text" value="All"/> <input type="checkbox"/> Include subfolders
Species: <input type="text"/> <a href="#">Select and Add</a>	Design Number: <input type="text"/> <a href="#">Upload</a>
Design Status: <input checked="" type="radio"/> All <input type="radio"/> Active <input type="radio"/> Obsolete	Created By: <input type="text"/>
Created Date: From: <input type="text"/> To: <input type="text"/>	Containing probe group: <input type="text"/> <a href="#">Select and Add</a>
Keywords: <input type="text"/>	

[Search](#) [Reset](#)

Design Format:  Status:  [Filter](#)

Search Results: 1 matching results found

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